


Designation	Desinfectant Wipes Medibase																										
Reference(s)	The box of 100 wipes	3131820																									
	The refill bag of 100 wipes	3131821																									
Product picture																											
Product description	Wipes impregnated with a disinfectant solution for cleaning and disinfecting non-invasive and non-submersible medical devices (small medical and dental equipment).																										
Characteristics and composition	<p>Less than 5% non-ionic surfactants, perfumes including limonene, disinfectants: Ethanol, Alkyl, Dimethylbenzylammonium chloride, Propan-2-ol.</p> <p>Wipes impregnated with a disinfectant solution for cleaning and disinfecting non-invasive and non-submersible medical devices (small medical and dental equipment).</p> <ul style="list-style-type: none"> - Wipes particularly large, thick, strong and sweet. - Less than 5% non-ionic surfactants, perfumes including limonene, disinfectants: Ethanol, Alkyl, Dimethylbenzylammonium chloride, Propan-2-ol. 																										
	<table border="1"> <thead> <tr> <th colspan="2">ACTIVITE MICROBIENNE</th> <th>Contact time</th> </tr> <tr> <th colspan="3">General treatments</th> </tr> </thead> <tbody> <tr> <td rowspan="2">Bactericidal</td> <td>EN13727* - EN14561</td> <td>5min</td> </tr> <tr> <td>EN14561: MRSA</td> <td>5min</td> </tr> <tr> <td>Fungicidal</td> <td>EN14562: Aspergillus fumigatus</td> <td>5min</td> </tr> <tr> <td>Yeasticidal</td> <td>EN13624 - EN 14562: Candida albicans</td> <td>5min</td> </tr> <tr> <td>Tuberculocidal</td> <td>EN14348 - EN 14563 : Mycobacterium terrae</td> <td>15min</td> </tr> <tr> <td rowspan="2">Virucidal</td> <td>EN14476* : HBV - HCV - Adenovirus Coronavirus - Novovirus - VRS - H1N1 - HSV1</td> <td>1min</td> </tr> <tr> <td>EN14476: Polyomavirus</td> <td>1min</td> </tr> </tbody> </table>		ACTIVITE MICROBIENNE		Contact time	General treatments			Bactericidal	EN13727* - EN14561	5min	EN14561: MRSA	5min	Fungicidal	EN14562: Aspergillus fumigatus	5min	Yeasticidal	EN13624 - EN 14562: Candida albicans	5min	Tuberculocidal	EN14348 - EN 14563 : Mycobacterium terrae	15min	Virucidal	EN14476* : HBV - HCV - Adenovirus Coronavirus - Novovirus - VRS - H1N1 - HSV1	1min	EN14476: Polyomavirus	1min
ACTIVITE MICROBIENNE		Contact time																									
General treatments																											
Bactericidal	EN13727* - EN14561	5min																									
	EN14561: MRSA	5min																									
Fungicidal	EN14562: Aspergillus fumigatus	5min																									
Yeasticidal	EN13624 - EN 14562: Candida albicans	5min																									
Tuberculocidal	EN14348 - EN 14563 : Mycobacterium terrae	15min																									
Virucidal	EN14476* : HBV - HCV - Adenovirus Coronavirus - Novovirus - VRS - H1N1 - HSV1	1min																									
	EN14476: Polyomavirus	1min																									

		EN14476: Rotavirus	15min
	Additional uses		
	Bactericidal	EN1276	30s
		EN13697	5min
	Fungicidal	EN13697: Aspergillus niger	5min
	Yeasticidal	EN1650* - EN 13697: Candida albicans	5min
	*In dirty conditions		
Packaging	- The box of 100 wipes The refill bag of 100 wipes		
Shelf life	3 month from the day of opening		
Indications of use	<div style="border: 1px solid black; padding: 5px; width: fit-content; margin-bottom: 10px;">LOT</div> <p>Ready-to-use wipes. Single-use wipes. When using the box for the first time:</p> <ol style="list-style-type: none"> 1. Remove the cover, 2. Permanently remove the seal, 3. Thread the central wipe through the hole in the cover with clean hands and 4. Replace the cover. 5. Use the wipe on the equipment. 6. Leave on for 30 seconds to 15 minutes according to the antibacterial activity required. 7. Leave to dry in the open air. 8. Do not rinse. 9. Close the flap after each use to prevent the wipes from drying out. 		
Legal information	Medical Device of class IIA CE Marking 0459. G-MED Notified body. Manufacturer : FELT S.A		