
Protective sleeves for LED lamps without optic fibre

Universal Camera Sleeves

Suitable for all commercially available camera systems

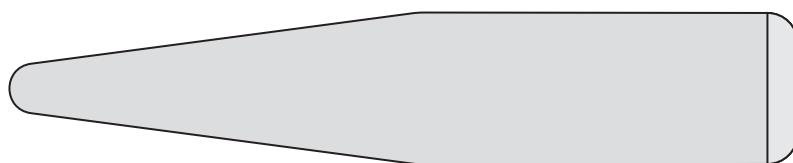
Dimensions 5.6 x 23.5 cm

Packaging unit 200 pieces

REF 3146065

- 1 Pull the Universal Camera Sleeve as far as possible over the intraoral camera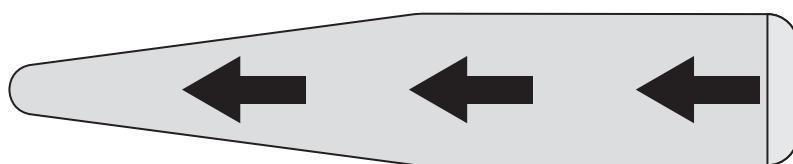

- 2 Dispose of properly after use

Intended use

Protective sleeves for LED lamps without optic fibre are single-use, transparent plastic protective sleeves to prevent crosscontamination between patients and to protect the intraoral camera during digital dental radiographs. The protective sleeves are compatible with the most common handheld intraoral cameras on the market.

Performance data

Protective sleeves for LED lamps without optic fibre are made out of latex-free, odorless plastic and the material was tested not to provoke skin irritation.

The sleeves do not affect the performance of the camera systems (lenses).

The sleeves serve as a support for the dental hygiene measures but do not replace manufacturer specifications for the disinfection or sterilization of the reusable intraoral sensor.

Safety precautions*

- The medical device may only be used by (trained) dental personnel.
- The protective sleeves are to be used exclusively with new disposable gloves, this avoids cross-contamination.
- Use a new hygienic protective sleeves each time you use the sleeves.
- Only use hygienic protective sleeves that are suitable for camera systems.
- Hygienic protective covers are disposable and must not be reused under any circumstances.
- In case of multiple use, cross-contamination between patients may occur, this may lead to infections.
- Dispose of the hygienic protective cover after each patient/use.
- Never use a damaged hygienic protective cover.
- Easy application and removal according to the illustrations overleaf.

All serious incidents related to the product must be reported to the manufacturer and the competent authority of your country.

Symbol designation – Explanation of the symbols used on packaging and labelling according to EU medical device standard DIN EN ISO 15223-1:2022-02

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M.G. Kurth GmbH